

ez-Track¹

Rapid Fluorescent Assay for the TDM of Biologics*

Simple

- 1 step to result

Fast

- 10 min for drug measurement
- 12 min for ADA_b measurement

Flexible

- Whole blood (Capillary / Venous)**
- Serum
- Plasma

Reliable

- Excellent correlation with i-Tracker[®] and LISA TRACKER[®] reference kits
- Calibrated on NIBSC/WHO international standards**
- Validated on princeps and biosimilars

* Therapeutic Drug Monitoring

** depending on test: refer to the instruction for use

Please check regulatory status in your country.

Not for in vitro diagnostic use in Australia unless first included in the Australian Register of Therapeutic Goods.

ez-Track¹



Time Resolved Fluorescence technology

Connectable to the LIS


Friendly user interface

Compact and lightweight

Simple and quick test procedure


Serum / Plasma

①

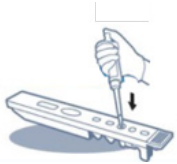


Dispense 100 µL of sample (serum or plasma) on the sample well

②

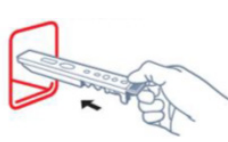

Insert the cartridge into the holder

③

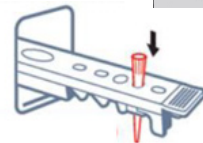

Insert the C-tip or a tip provided into the tip well. Press « START »


④



The result is obtained in 10 to 12 minutes

Whole blood

①

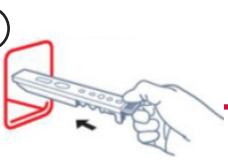


Insert the cartridge into the holder

②



Collect capillary blood using C-tip (provided in the kit)

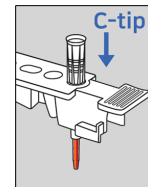

MEASUREMENT RANGE

Drug

Infliximab 0,2 – 50 µg/mL

Adalimumab 0,2 – 50 µg/mL

Golimumab 0,2 – 30 µg/mL

Anti-Drug Antibodies

Anti-Infliximab 4 – 250 AU/mL

Anti-Adalimumab 3 – 200 AU/mL

Anti-Golimumab 2 – 40 ng/mL

Anti-Infliximab Total Ab 8 – 250 AU/mL

ez-Tracker

Reference

ETx 002-24

ETx 003-24

ETI 003T-24

ETx 002-C / ETx 003-C

ETx 002-CAL / ETx 003-CAL

Designation

ez-Tracker Drug

ez-Tracker Anti-Drug Antibodies

ez-Tracker Anti-Infliximab Total Ab

ez-Tracker Drug / ADAb Controls

ez-Tracker Drug / ADAb Calibrators

Packaging

24 tests

24 tests

24 tests

2 x 1 mL

2 x 1 mL

x = Infliximab / Adalimumab / Golimumab / Vedolizumab / Ustekinumab / Etanercept*

*Available soon



Boditech (Republic of Korea)

14 rue Ambroise Croizat
77183 Croissy Beaubourg
France



Pictures may differ from actual products.

Phone : +33 (0) 1 64 62 10 12

Fax : +33 (0) 1 64 62 09 66

info@theradiag.com

www.theradiag.com

