

Classification criteria

# EliA testing for antiphospholipid syndrome (APS) in relation to the new 2023 APS classification criteria

# Why did EULAR/ACR update the APS classification criteria in 2023?<sup>1</sup>

- There were no evidence-based definitions incorporated in the revised 2006 criteria
- New criteria ensure high-quality, risk-stratified epidemiologic studies and clinical trials
- Maximizing specificity can help to better understand disease pathophysiology and treatment effects

# Why are the new APS classification criteria relevant for you and your lab?

- The classification criteria are intended to classify patients for medical research purposes<sup>2</sup>
- While the classification criteria were not developed for diagnosing APS, for facilitating and standardizing clinical research, they are often used as diagnostic criteria in the clinical setting<sup>2</sup>
- Lab professionals often have only limited information about patient samples, not knowing if they are used for clinical studies or patient diagnosis



# What are the key changes in the new classification criteria?1

- Recommend delaying the use of the new automated platforms (chemiluminescence immunoassay, multiplex and flow cytometry) and instead using standardized enzyme-linked immunosorbent assay (ELISA) to determine antiphospholipid (aPL) antibodies
- Evaluate in combination both IgG anti-cardiolipin (aCL) and IgG anti-B2-glycoprotein (aB2GPI) positivity
- Separately measure aCL/aB2GPI IgG and IgM isotypes
- 2 levels of aCL/aβ2GPI positivity: moderate and high positive results based on ELISA technologies





#### 2023 new APS classification criteria workflow<sup>1</sup>

#### Entry criteria includes

At least one clinical criterion **PLUS** 

A positive aPL test within three years of clinical criterion

Clinical domain

### Laboratory domain (aPL test and criteria)

Score		Score
	aPL by solid phase assay (persistent)	
1	Moderate or high positive IgM (aCL and/ or aβ2GPI)	1
5	Moderate positive IgG (aCL and/or aβ2GPI)	4
	High positive IgG (aCL or aβ2GPI)	5
	High positive IgG (aCL and aβ2GPI)	7
	1	aPL by <b>solid phase</b> assay (persistent)  1 Moderate or high positive IgM (aCL and/ or aβ2GPI)  5 Moderate positive IgG (aCL and/or aβ2GPI)  High positive IgG (aCL or aβ2GPI)

#### Total score

if the sample met at least 3 points from clinical plus 3 points from laboratory domain

it is classified as APS for research purposes

# Why are EliA™ aPL tests considered to be standardized ELISA-based assays?

- EliA aPL tests follow the same principle and detection system as ELISA5-8
- EliA aPL tests are indirect ELISA assays using fluorescence instead of colorimetry as the detection method<sup>5-8</sup>
- Previous publications reported the equivalence between commercial ELISA and FEIA method used in EliA APS assays 3



# How can our EliA aPL tests help you to maintain compliance with the new criteria?

- EliA aPL tests are indirect ELISA assays as part of the standardised ELISA testing recommended in the new criteria.
- EliA aPL tests include aCL, aβ2GPI, and both IgG and IgM isotypes for a full offering



## For classification (medical research) purposes

	aCL IgG	aCL IgM	aβ2GPI lgG	aβ2GPI IgM
Unit	ELISA units	ELISA units	ELISA units	ELISA units
Moderate Positive	40 - 79	40 - 79	40 - 79	40 - 79
High Positive	> 80	> 80	> 80	> 80

## For diagnostic (clinical) purposes

	EliA Cardiolipin IgG	EliA Cardiolipin IgM	EliA β2-Glycoprotein IgG	EliA β2-Glycoprotein IgM
Unit	GPL-U/ml	MPL-U/ml	EliA U/ml	EliA U/ml
Negative	< 10	< 10	< 7	< 7
Equivocal	10 - 40 (weak positive)	10 - 40 (weak positive)	7 - 10	7 - 10
Positive	> 40	> 40	> 10	> 10

1. Barbhaiya M. (2023) ACR/EULAR Antiphospholipid Syndrome Classification Criteria. Arthritis Rheumatol 75(10):1687-1702. 2. Vandevelde, A., & Devreese, K. M. J. (2022). Laboratory Diagnosis of Antiphospholipid Syndrome: Insights and Hindrances. Journal of clinical medicine, 11(8), 2164. 3. Mattia, E, et al. (2015) A contribution to detection of anticardiolipin and anti-β2glycoprotein I antibodies: Comparison between a home-made ELISA and a fluorescence enzyme immunoassay. Clin Chim Acta. 15(446):93-6. 4. Villalta, D, et al. (2009) Accuracy of the first fully automated method for anti-cardiolipin and anti-beta2 glycoprotein I antibody detection for the diagnosis of antiphospholipid syndrome. Ann N Y Acad Sci. 1173:21-7. 5. EliA β2-Glycoprotein I gG Directions for Use. (2023) 250-5532-023 / UK. 8. EliA β2-Glycoprotein I IgM Directions for Use. (2023) 250-5530-023 / UK. 9. Crowther, J.R. (1995). Chapter 2 Basic Principles of ELISA.



Learn more at thermofisher.com/phadia

© 2024 Thermo Fisher Scientific Inc. All rights reserved. All trademarks are the property of Thermo Fisher Scientific and its subsidiaries unless otherwise specified. 22975 EN 385253.AUTO.EN.v1.24

thermo scientific

